


TREATMENT FOR THE LOWER EXTREMITY

The Flexitouch® system, a fully programmable segmented pneumatic compression device with calibrated gradient pressure, provides a variety of pneumatic compression therapy options to meet the lymphedema patient's needs. The only pneumatic compression device clinically proven to improve lymphatic function¹, the Flexitouch pressure profiles are easily tolerated and demonstrated to be effective^{2,3}, resulting in improved outcomes and treatment compliance.

LOWER EXTREMITY TREATMENT SOLUTION

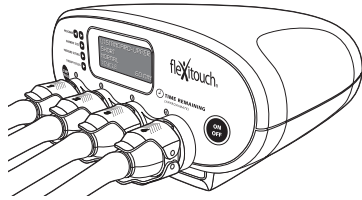

Controller Unit (E0652)

The controller unit is a calibrated sequential pneumatic device generating compressed air which is delivered via hoses to multi-chamber garments. The controller unit is programmable to address the desired therapeutic objectives.

Full Leg Garment (E0667) Calf-Foot Garment (E0669)

The garments consist of an inner layer of polyurethane cells enclosed in a soft stretchable nylon outer layer and are adjustable using hook and loop closures for a contoured fit. The garments consist of up to 24 chambers which are each 1-1/2 inches wide.

Optional accessories are available to meet specific patient needs. Accessories, which may include a trunk accessory, waist extender or closure strap, are included at no extra charge.

Product Specifications

FDA Status:	Cleared to market (Class II in July 2002)	
Controller Unit:	Weight	8 pounds
	Dimensions	10.4" x 9.8" x 4.7"
Garment Fabric:	100% nylon (latex free)	

TREATMENT PROGRAM OPTIONS

The Flexitouch's programmability allows the clinician to choose from a number of treatment configurations. Flexibility in programming means the Flexitouch can provide traditional pneumatic therapy, with the added benefit of specific programs to address problem areas.

Primary Programs



Full Leg

Program L8
45 Minutes

Distal to proximal drainage with full leg garment.*



Full Leg Plus

Program L4
45 Minutes


Distal to proximal drainage preceded by proximal clearing with full leg garment.*



Full Leg and Core

Program L1
60 Minutes


Distal to proximal drainage preceded by proximal clearing with full leg garment* and trunk accessory.

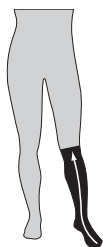

Half Leg

Program L7
30 Minutes

Distal to proximal drainage with half-leg garment.

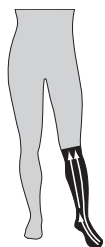

Half Leg Plus

Program L5
30 Minutes

Distal to proximal drainage preceded by proximal clearing with half-leg garment.

Supplemental Programs



Trunk

Program L2
15 Minutes



Trunk and Thigh

Program L3
30 Minutes



Foot Only

Program L6
15 Minutes

*Full leg garment will be replaced with calf-foot garment and trunk-thigh accessory for EXT sized patients.

1. Kristen E. Adams, John C. Rasmussen, Chinmay Darne, I-Chih Tan, Melissa B. Aldrich, Milton V. Marshall, Caroline E. Fife, Erik A. Maus, Latisha A. Smith, Renie Guillod, Sunday Hoy, and Eva M. Sevick-Muraca. *Biomedical Optics Express*. 2010; Vol. 1(1): 114–125 2. Caroline E. Fife, MD, Suzanne Davey, OTR/L CLT-LANA, Erik Maus, MD, Renie Guillod, MD, and Harvey N. Mayrovitz, PhD. *Supportive Care in Cancer*. 2012; Vol. 20(12): 3279–3286 3. Satish C. Muluk, MD, Alan T. Hirsch, MD, and Elise C. Taffe, MD. *European Journal of Vascular and Endovascular Surgery*. 2013; Vol. 46(4): 480–487